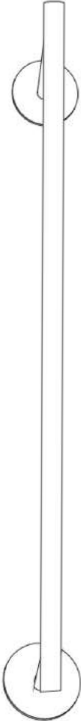
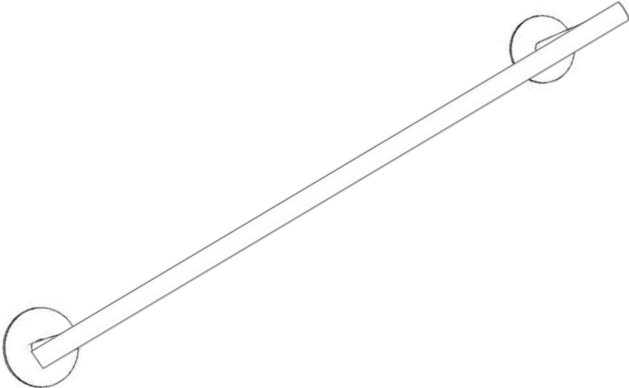
HORIZONTAL POSITION

VERTICAL POSITION

ANGLE POSITION

**LINDEN ADA GRAB BAR**

## ⚠ ENGLISH

Do not dispose of any packaging or contents of the shipping carton until assembly is completed to avoid accidentally discarding small parts or hardware.

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### Care Instructions:

Dust often using a clean, soft, dry and lint-free cloth.  
Blot spills and wipe with a clean, damp cloth immediately.  
We do not recommend the use of chemical cleansers, abrasives or furniture polish on our lacquered finish.  
Hardware may loosen over time. Periodically check to make sure all connections are tight. Re-tighten if necessary.

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### SAFETY INSTRUCTIONS:

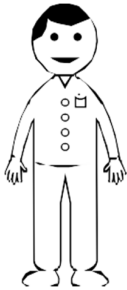
At least one side of the bar must be attached to a wall stud.  
Based on standard stud size, 2 of the 3 screws on 1 end of the bar must be mounted onto the wall stud'  
Product is recommended to be mounted on a 5/8 inch wall.  
The grab bar can be mounted at horizontally, vertically or at an angle.  
Mounting MUST be done by a certified professional only.  
Mounting location MUST be verified by a qualified professional to provide safety and function.  
When in use by disabled customers, another person needs to be around to supervised or assist.  
Grab bars are meant for assistance purpose only and not to hold/support the entire weight of any individual.

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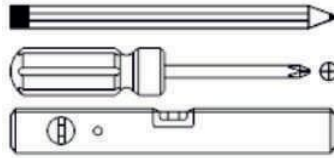
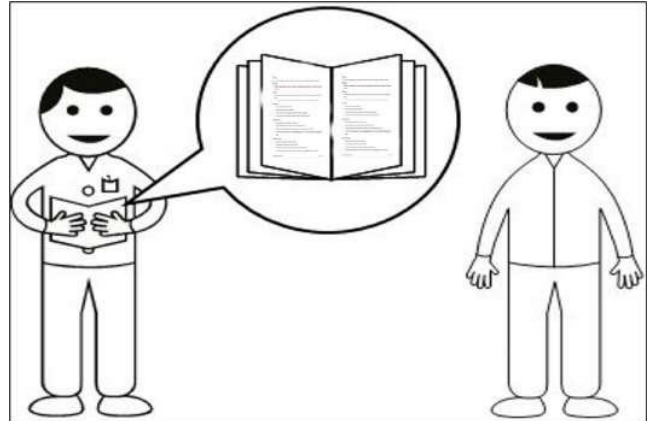
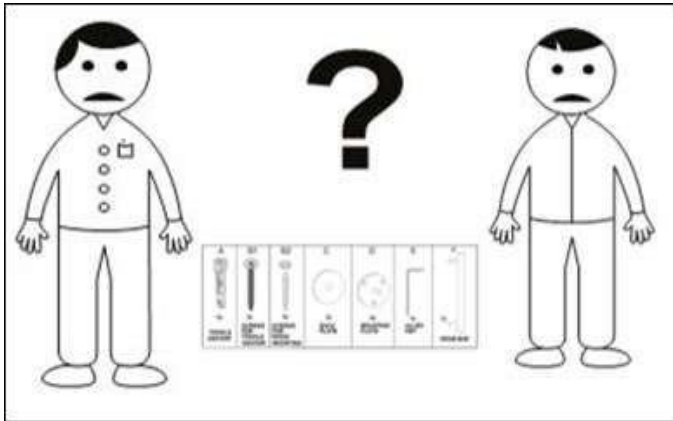
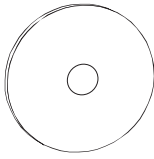
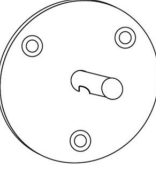
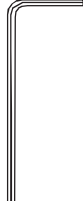
**X2**

NOT INCLUDED / NON INCLUS / NO INCLUIDAS

<p><b>A</b></p>  <p><b>6x</b></p> <p><b>TOGGLE ANCHOR</b></p>	<p><b>B1</b></p>  <p><b>6x</b></p> <p><b>SCREWS FOR TOGGLE ANCHOR</b></p>	<p><b>B2</b></p>  <p><b>3x</b></p> <p><b>SCREWS FOR WOOD MOUNTING</b></p>	<p><b>C</b></p>  <p><b>2x</b></p> <p><b>BACK PLATE</b></p>	<p><b>D</b></p>  <p><b>2x</b></p> <p><b>MOUNTING PLATE</b></p>	<p><b>E</b></p>  <p><b>1x</b></p> <p><b>ALLEN KEY</b></p>	<p><b>F</b></p>  <p><b>1x</b></p> <p><b>GRAB BAR</b></p>
--	--	--	---	--	--	---