MERCER ADA GRAB BAR

ENGLISH

- One side of the bar must be attached to a wall stud.
- Based on standard stud size, 2 of the 4 screws on 1 end of the bar must be mounted onto the wall stud.
- Product mounted on a 5/8 inch wall for contract grade customers.
- Instructions for mounting the grab bar horizontally, vertically or at an angle.
- Mounting must be done by a certified professional.
- Mounting location needs to be verified by a qualified professional to provide safety and function.
- When in use by disabled customers, another person should be around to supervised or assist.
- Grab bars are meant for assistance purpose only and not to hold/support the entire weight of any individual.
- DO NOT install this product without first reading and understanding this instruction sheet. If you are unable to understand these Warnings and Instructions, contact a professional or technical personnel for assistance before attempting to install this product - otherwise, injury may occur.
- After ANY adjustments, repair or service and BEFORE use, make sure that all attaching hardware is tightened securely.
- Assist Bars add security for bath and commode areas if attached as instructed. Location, size, and angle of bars should be determined by a qualified professional to provide optimum safety and function.
- Use EXTREME caution on wet tub or floor surfaces.
- Be sure Assist Bars are correctly and securely installed. NEVER use Assist Bars if any looseness is noted.

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Care Instructions:

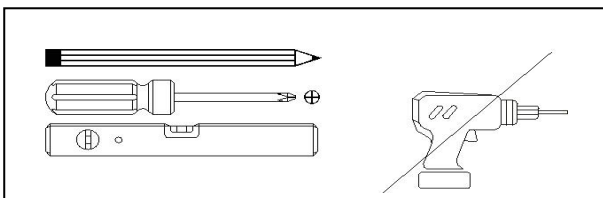
- Dust often using a clean, soft, dry and lint-free cloth.
- Blot spills and wipe with a clean, damp cloth immediately.
- We do not recommend the use of chemical cleansers, abrasives or furniture polish on our lacquered finish.
- Hardware may loosen over time. Periodically check to make sure all connections are tight. Re-tighten if necessary.

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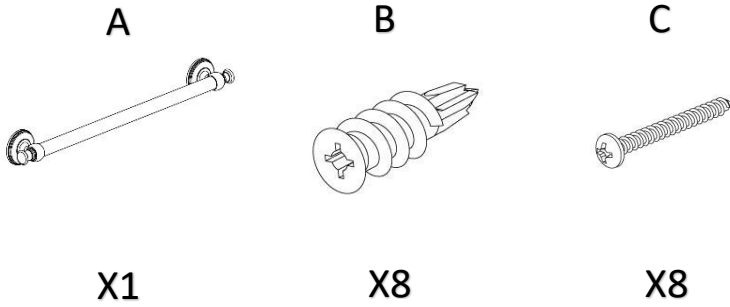
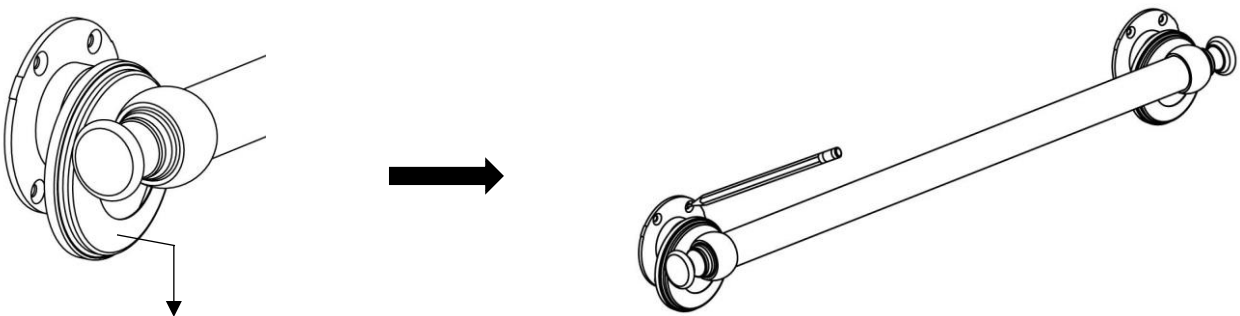
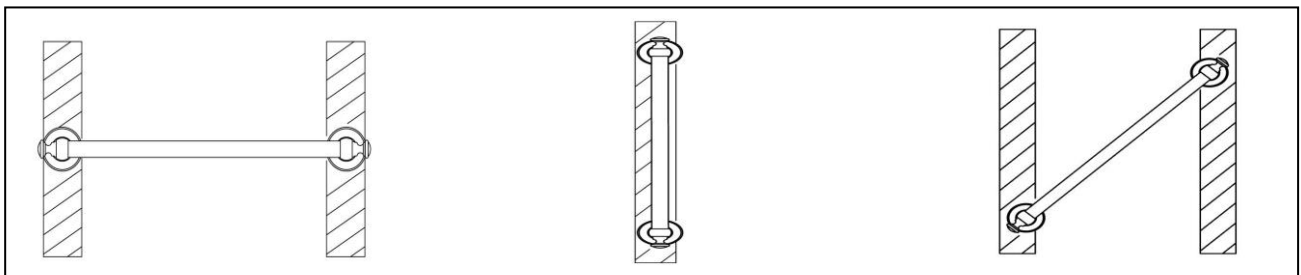
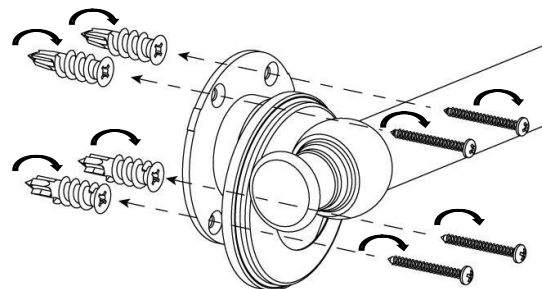
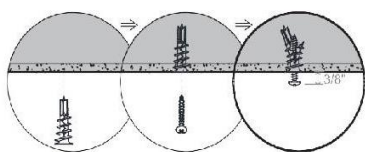
NOT INCLUDED / NON INCLUS / NO INCLUIDAS

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